



HDMag™ Control Panel Reference Card

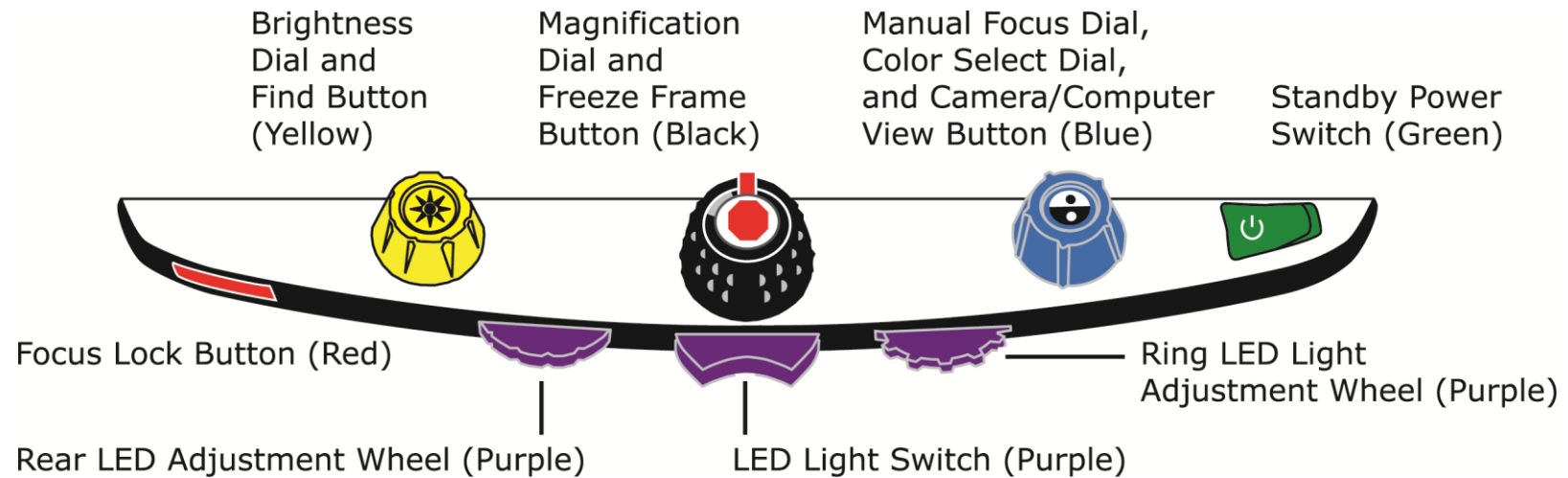

Dial / Button / Switch	Functionality
Brightness Dial / Find Button (Yellow)	<ul style="list-style-type: none"> • Turn to increase or decrease screen brightness • Press and hold center to zoom out using the Find feature. Release to zoom in.
Magnification Dial / Freeze Frame Button (Black)	<ul style="list-style-type: none"> • Turn to increase or decrease magnification • Press center to turn Freeze Frame on or off
Manual Focus Dial / Color Select Dial / Camera and Computer View Button (Blue)	<ul style="list-style-type: none"> • Turn to adjust manual focus. Focus Lock  (red button) must be on. • Turn to choose a color mode that best suits your vision needs • Press center to switch between the HDMag camera and an attached computer • Press center to switch the single horizontal reference line on or off if not connected to a computer

Dial / Button / Switch	Functionality
Standby Power Switch (Green)	<ul style="list-style-type: none"> • Press to the right to place the unit in standby mode. Press to the left to turn on the unit.
Focus Lock Button (Red)	<ul style="list-style-type: none"> • Press to turn on Focus Lock. Press again to turn off Focus Lock. Note: Focus Lock must be on to use manual focus.
LED Light Switch (Purple)	<ul style="list-style-type: none"> • Press to the right to turn on or off the ring LED light that is located around the camera window • Press to the left to turn on or off the rear LED lights mounted on the side of the unit
Rear LED Light Adjustment Wheel (Purple)	<ul style="list-style-type: none"> • Turn to increase or decrease rear LED light brightness
Ring LED Light Adjustment Wheel (Purple)	<ul style="list-style-type: none"> • Turn to increase or decrease ring LED light brightness